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
Customer: KIND  
Ref manual : KIND4 GAM-131\_A01  
Open format : Width 310 x Height 180 mm  
Closed format : Width 155 x Height 60 mm  
Folding: 3  
Brand: KIND  
Date: 12/10/2023

Print type: Offset or digital  
Print media: White paper 70 gsm offset  
Software used: In-Design CC 2023


Colors used:

-  Cyan
-  Magenta
-  Yellow
-  Black


Protective varnish: None  
Selective varnish: None

Texts specifications:  
Regulatory texts modified by the customer

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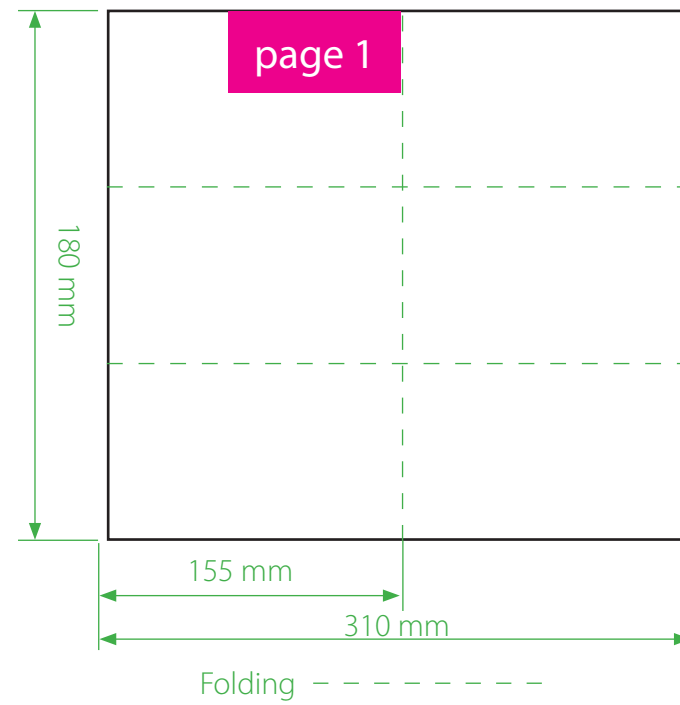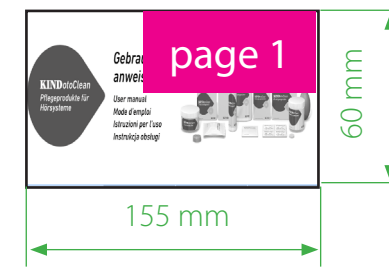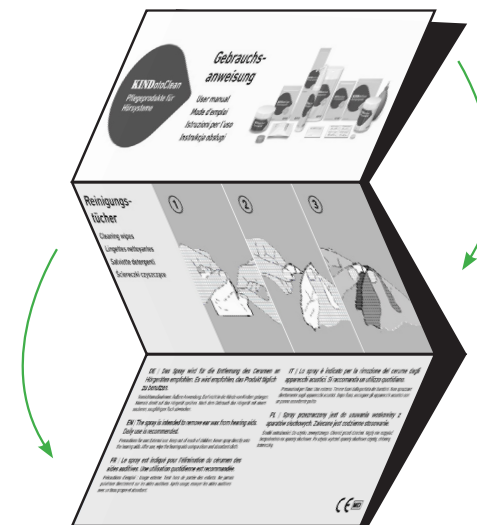
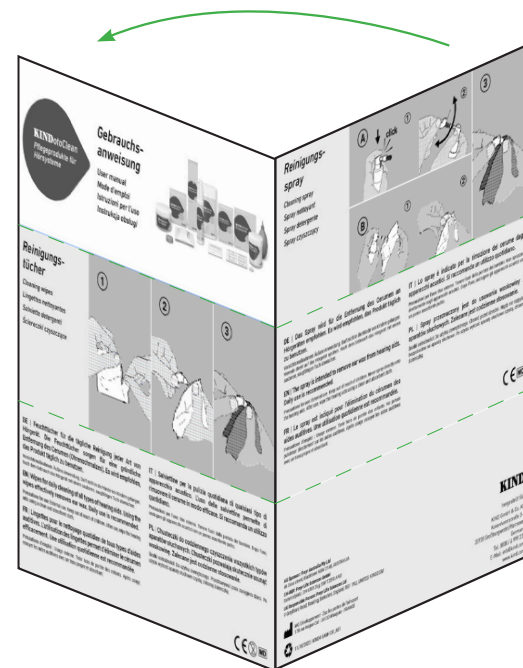
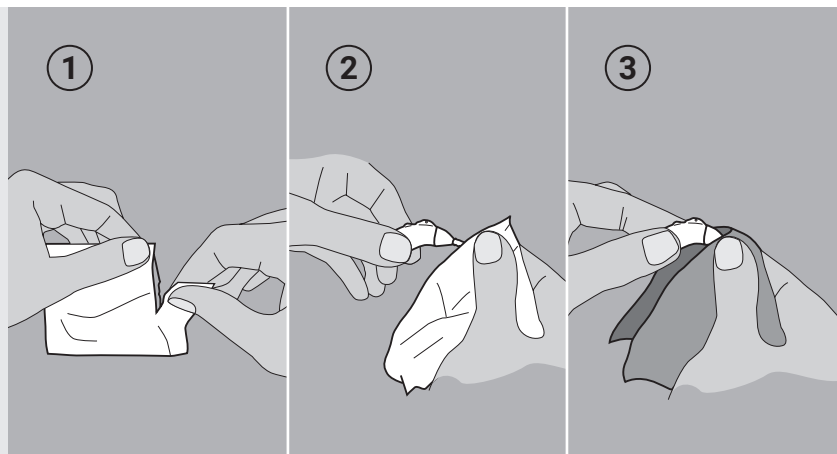
**EN** | Wipes for daily cleaning of all types of hearing aids. Using the wipes effectively removes ear wax. Daily use is recommended.

**Precautions for use:** External use. Keep out of reach of children. After use, wipe the hearing aids using a clean and absorbent cloth.

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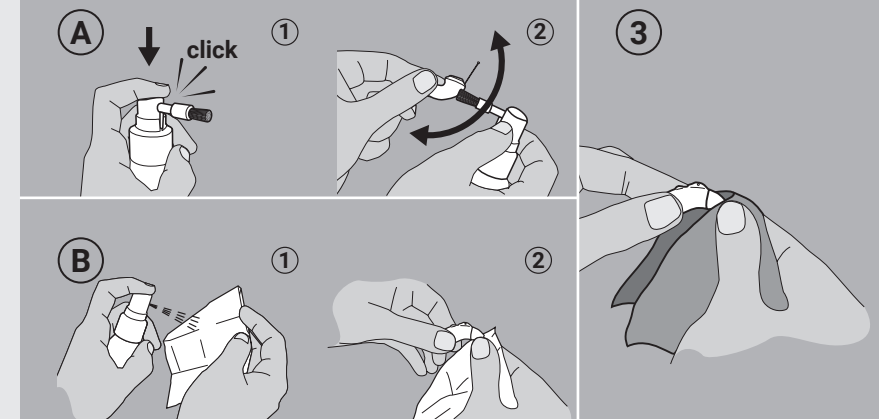
**EN** | The spray is intended to remove ear wax from hearing aids. Daily use is recommended.

**Precautions for use:** External use. Keep out of reach of children. Never spray directly onto the hearing aids. After use, wipe the hearing aids using a clean and absorbent cloth.

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**AU Sponsor:** Freyr Australia Pty Ltd  
46 Dora street, Blacktown NSW 2148, AUSTRALIA

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**UK Responsible Person:** Freyr Life Sciences Ltd  
9 Greyfriars Road, Reading, Berkshire, England, RG1 1NU, UNITED KINGDOM

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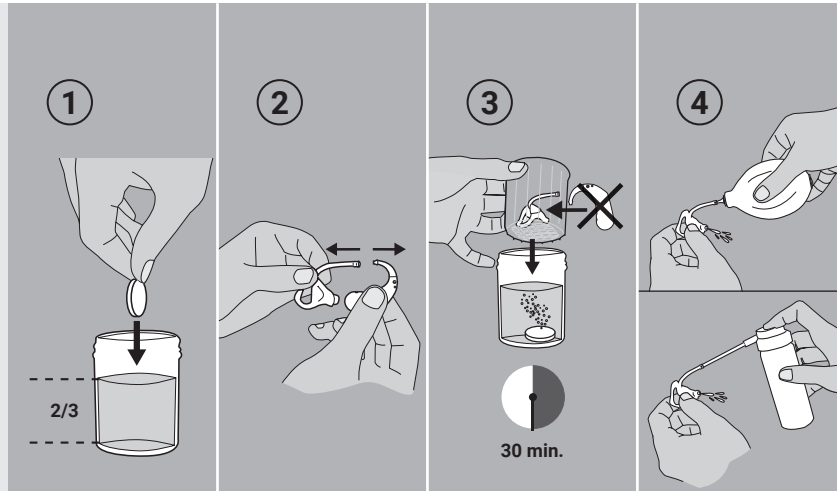
**EN |** Cleaning tabs for ear tips. 30 minutes are enough to effectively remove ear wax. Place the cleaning tab in the cup and fill it two thirds full of water. Dispose of the solution after use. Full cleaning of the ear tips is recommended once or twice per week.

**Precautions for use:** External use. Single use. Keep out of the reach of children. Harmful if swallowed. Causes severe skin burns and eye damage. Wash thoroughly after handling. Harmful to aquatic life – Avoid release to the environment. If medical advice is sought, keep the container or label on hand. If swallowed: Call a poison centre / a doctor. Rinse the mouth. Do not induce vomiting. If in contact with the eyes: carefully rinse with water for several minutes.

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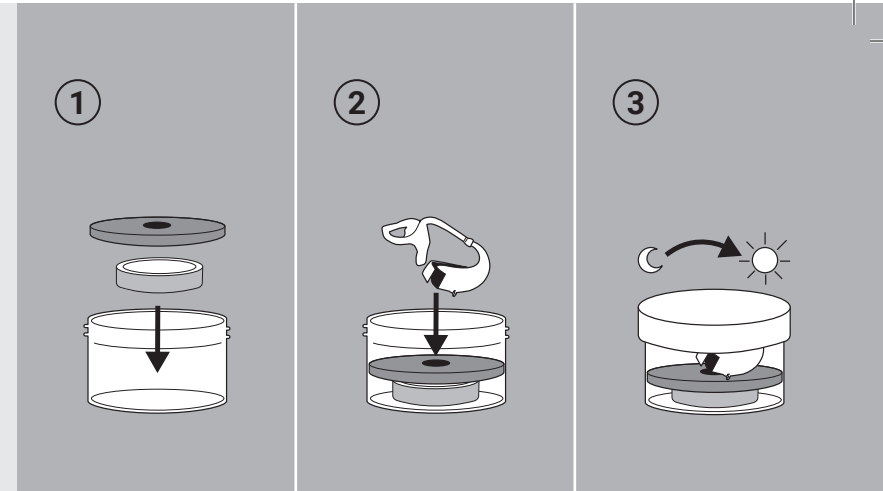
**EN |** Capsules to prevent damage to electronic components caused by humidity. Open the battery compartment if the hearing aid has one. Close the cup after each use. Daily use is recommended. Replace the capsule when it is almost white. The service life is about 1 month (drying capsules) / 3 months (drying capsules ultra-powerful).

**Precautions for use:** Do not pierce the capsule. Do not swallow. Keep out of reach of children.

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**EN | For the maintenance of hearing aids.** If you experience problems, contact a hearing aid specialist. All serious incidents related to the device should be the subject of a notification to the manufacturer and the competent authorities in the Member State in which the user is located.

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