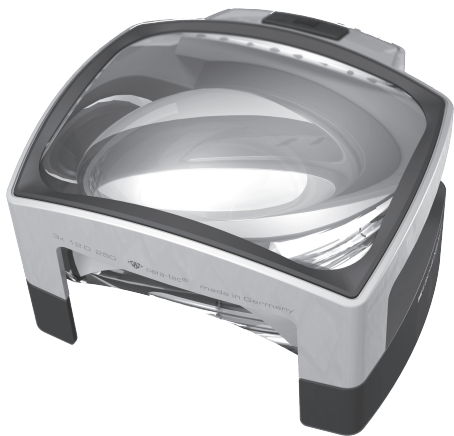
# visolux+

**Bedienungsanleitung**  
**User Manual**  
**Mode d'emploi**  
**Istruzioni d'uso**  
**Manual de instrucciones**  
**Gebruiksaanwijzing**  
**Betjeningsvejledning**  
**Bruksanvisning**  
**Käyttöopas**  
**Instrukcja obsługi**  
**Provozní návod**  
**Руководство по эксплуатации**  
**サービスマニュアル**

**ESCHENBACH**

Deutsch.....	3
English.....	8
Français .....	13
Italiano .....	18
Español.....	23
Nederlands .....	28
Dansk.....	33
Svenska .....	38
Norsk.....	43
Suomi.....	48
Polski.....	53
Česky.....	58
Русский.....	63
日本語 .....	68

## Deutsch

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## English

### **Prior to initial operation**

Before first use, read the manual carefully to learn all the functions of the device. visolux<sup>+</sup> is designed for use as optical stand magnifier that generates an enlarged image of the reading material.

### **Intended use**

This product is designed exclusively for providing enlarged images of text, pictures, or other object details.

### **Safety Instructions**

- ▶ Danger of glare and injury! Do not look directly at the light source on the inner part of the housing!
- ▶ Do not allow children to handle electronic devices without supervision.
- ▶ Check the device for visible damage. In case of any damage have the product checked by a qualified specialist before you use it again.
- ▶ Never open the housing of visolux<sup>+</sup>. There are no serviceable parts whatsoever inside. Should the housing be opened the warranty becomes invalid.

- ▶ Never expose visolux<sup>+</sup> to moisture.
- ▶ The product is a Class I medical device (non-sterile, without measuring function) and complies with Regulation (EU) 2017/745. For use of the product outside the EU member states, the respective country-specific directives must be observed.
- ▶ Protect visolux<sup>+</sup> from impacts, moisture and excessive heat. Never lay your magnifying light on a radiator or in direct sunlight.
- ▶ Inform other persons, particularly children, of these dangers!

### **Initial operation**

Before using the device for the first time, open the battery compartment (❶), insert the supplied batteries (3 x 1.5 V, size AA), observing the correct polarity into the battery compartment and close the battery compartment.

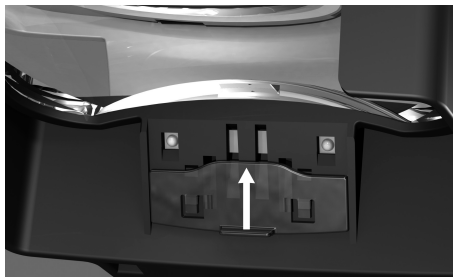
## Switching on/off

Press the on / off switch (❶) to turn the device on or off. To save battery power, the unit automatically switches off after about 30 minutes. If you wish to continue using it, turn it on again.

## Operation

Put visolux<sup>+</sup> on a flat surface for a magnified display of the reading material. Due to its large field of view, visolux<sup>+</sup> is also suitable for viewing pictures and postage stamps. If you think the colour temperature of the LED light is too cold, you can slide the built-in orange filter in front of the LEDs in order to achieve a warmer colour temperature.

## Care Instructions

- ▶ When cleaning visolux<sup>+</sup> do not use any soap solutions that contain softening agents, no alcoholic or organic solvents and no abrasive detergents! The lenses could be destroyed!
- ▶ Do not clean visolux<sup>+</sup> in an ultrasonic bath or under flowing water!
- ▶ Clean visolux<sup>+</sup> only with a soft dry cloth. For heavier soiling (e.g. fingerprints), moisten the cloth slightly.

## **Specifications**

Item no:	1566
Magnification:	3x, 12 dpt
Lens size:	100 × 75 mm
Colour temperature:	6500 K / 3400 K with filter

## **Warranty**

Within the parameters of the applicable statutory provisions, we provide a warranty for the functioning of the product described in this manual with regard to faults arising in connection with manufacturing defects or material defects. In event of damage due to improper handling, including damage from a fall or impact, no warranty claims can be made. Warranty claims are only accepted upon presentation of the sales receipt!

## Disposal

- ▶ This product is subject to the European WEEE Directive on waste electrical and electronic equipment in its currently valid version. Dispose of your product via a certified disposal facility or your municipal disposal facility, not in the household waste. Observe the rules applicable in your country. In case of doubt, contact your disposal facility. Dispose of all packaging materials in an environmentally friendly manner.
- ▶ Batteries may not be thrown out in the household waste or be incinerated, but must be disposed of according to local regulations for disposal of hazardous waste. Use the existing collection points.

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Eschenbach Optik GmbH